


Infant transport mattress

Instructions For Use




Activation in three easy steps

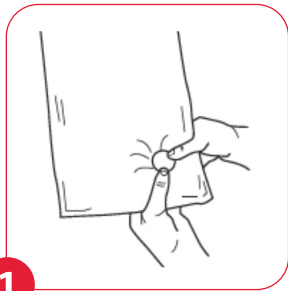

1

Grasp disc with the thumbs and forefingers of both hands. Bend disc back and forth until crystals form.

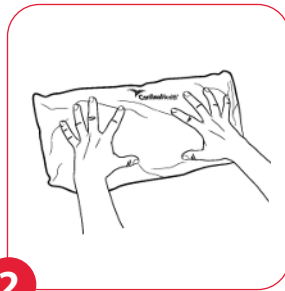

2

Knead mattress for 10 to 30 seconds to evenly distribute heat.



3

Indent center with palm to form small nest. **Lay infant in nest** on the fabric side of the mattress.

Periodically check on infant's skin and monitor core temperature.

Dispose into trash after single use.

For single use only. Applies to Cardinal Health™ Infant Transport Mattress Catalog Number 11475-010B.

Contents: Food grade sodium acetate and water. All contents are non-toxic.

Caution: U.S. Federal law restricts this device to sale by or on the order of the physician. (Rx Only). Use carefully with patient that has poor circulation. Consult a physician when using this product in non-transport environment.

Warning: For external use only. Monitor the infant's core temperature and check infant's skin for redness. Do not puncture or strike pack. Do not wrap the device with any additional heat source, such as a warm washcloth. Do not microwave. Do not heat externally. Do not reuse device; it is intended for one-time use only. Using a reheated device could cause a burn. All contents are non-toxic; however, if contents are in contact with eyes flush them thoroughly with water and immediately notify the healthcare provider. Do not swallow. If Infant Transport Mattress contents leak and come in contact with the skin, wash immediately with water and mild soap. Change affected clothing and bedding. Do not leave infant unattended. Do not use this Infant Transport Mattress on injured or unclothed skin. Use extra care on a premature infant. Activate away from face and infant. Use carefully. May cause serious burns. Do not use over sensitive skin areas or in the presence of poor circulation. The unattended use of the Infant Transport Mattress by children or incapacitated persons may be dangerous. Consult a physician when using this product in non-transport environment.

Storage and Handling: Do not use knife to open. Store at room temperature. Protect from excessive heat. Do not pre-heat, reuse by re-heating or freeze.

