

Infant heel warmer


Instructions For Use




Activation in three easy steps



1 Locate inner fluid bag and **squeeze firmly** to break.

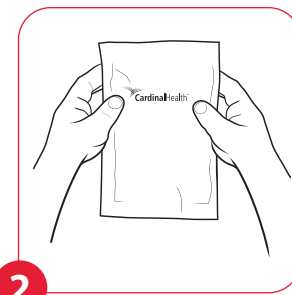

2 **Knead pack** to activate throughout.

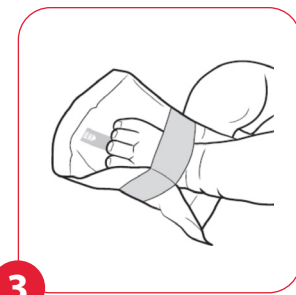

3 **Apply** to heel and secure with adhesive tab.

Hold warmer on the heel for 1-3 minutes, checking every 30 seconds, until the blood vessels dilate.

This makes blood sampling easier.

Dispose into trash after single use.

For single use only. Applicable for Cardinal Health™ Squeeze Activated Infant Heel Warmer Catalog Numbers 11460-010T, IHW4X4GVT and NOVAPLUS® V11460-010.

Contents: Food grade sodium acetate and water. All contents are non-toxic.

Caution: Do not use this Infant Heel Warmer on injured skin. Use with extra care on a premature infant. For external use only. Do not puncture or strike pack. Reaches a temperature of 104°F (from 75°F starting temperature). Do not microwave. Do not heat externally. Do not reuse a device; it is intended for one-time use only. Using a reheated device could cause a burn. If contents come in contact with eyes, flush them thoroughly with water and immediately notify the healthcare provider. If heel warmer contents leak and come in contact with skin, wash immediately with mild soap and water. Change affected clothing and bedding. Activate away from face and infant.

Storage and Handling: Do not use knife to open packaging. Store at room temperature. Do not freeze. Do not pre-heat, reuse by re-heating, or freeze.

