



CLINITEST hCG Test

CLIA-
waived

Automated and Connected

CLINITEST® hCG Pregnancy Test and CLINITEK Status® Family of Analyzers

Bring your hCG urine pregnancy testing to the next level with a simple and automated solution to standardize the testing process. Achieve speed, convenience, and confidence in test results unmatched by visual reading.

Eliminates error sources

- Accurately times the test for you—no watching the clock!
- Automates and removes subjectivity of result interpretation.
- Automatically transmits test results to minimize transcription errors.

Improves workflow efficiency

- Frees staff for patient care with true walkaway operation—add sample and the analyzer does the rest.
- Results in 5 minutes or less—no waiting for lab results.
- Optional bar-code scanner saves time, simplifies data entry, and eliminates transcription errors.

Provides fail-safe measures

- Proprietary Auto-Checks® technology on Multistix® Reagent Strips prevents testing if expired materials are used or quality-control testing fails or is not performed.
- Provides a unique flag for results in the “indeterminate” range where confirmatory testing is important for interpreting the significance of low-level hCG in qualitative tests.
- Operator lockout prevents use by unauthorized users.

Improves data capture and traceability

- Device connectivity provides:
 - 100% data capture
 - Automatic result recording and transmission to EMR/ EHR or LIS/HIS
- Audit-ready—each test result is linked to patient ID, operator ID, material lot number, and expiration date.

Pregnancy testing is simpler and more reliable with the CLINITEST hCG test.

siemens-healthineers.us/clinitesthcg



SIEMENS
Healthineers

CLINITEK hCG pregnancy test

The CLINITEK hCG pregnancy test is a lateral-flow, qualitative test with the following key specifications:

- 200 µL urine sample size
- Test results as fast as 2 minutes for a positive test, and 5 minutes to confirm a negative test
- Room-temperature storage
- Sensitivity of ≥ 25 mIU/mL
- >99% clinical accuracy
- Reported results: negative, positive, and borderline*

hCG testing is as easy as 1-2-3



1

Insert the cassette into the analyzer.



2

Press the Start button.



3

Add entire contents of pipette.

Pipettes included with cassette purchase.

Ordering Information

Product Code	Description
10310618 (1760)	CLINITEK hCG test (25 tests)
10470849 (1797)	CLINITEK Status® Connect System, CLINITEK Status+® Analyzer, CLINITEK Status Connector, and bar-code reader
10379675 (1780)	CLINITEK Status+ Analyzer
10376323 (1790)	CLINITEK Status Connector
10484591	CLINITEK Status Upgrade Kit (includes connector base and bar-code reader)
10282579	Bar-code Scanner

Gonadotropin, Chorionic (hCG), qualitative: CPT Code 84703

Flexible Connectivity Options

	Communication	Interfaces
CLINITEK Status+ Analyzer	Unidirectional	Siemens Healthineers data-management solutions; Telcor, RALS, EMRs via data-management software
CLINITEK Status Connect System	Bidirectional	Siemens Healthineers data-management solutions; Telcor, RALS, EMRs via data-management software

*Borderline test results are considered indeterminate and must be repeated within 48–72 hours.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
Phone: +49 9131 84-0
siemens-healthineers.com

USA

Siemens Healthcare Diagnostics Inc.
Point of Care Diagnostics
2 Edgewater Drive
Norwood, MA 02062-4637, USA
Phone: +1-877-229-3711
siemens-healthineers.us

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens Healthineers sales organization worldwide. Availability and packaging may vary by country and is subject to change without prior notice. Some/All of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features, which do not always have to be present in individual cases.

Siemens Healthineers reserves the right to modify the design, packaging, specifications, and options described herein without prior notice. For the most current information, please contact your local sales representative from Siemens Healthineers.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

Auto-Checks, CLINITEK, CLINITEK Status, CLINITEK Multistix, and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners. Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.