



QUIDEL®

# Sofia<sup>2</sup>




## Virena<sup>®</sup> Setup Instructions

### CUSTOMER SERVICE


*If Sofia 2 does not perform as expected, contact Quidel Technical Support 800.874.1517 (in the U.S.), 858.552.1100 (outside the U.S.), [technicalsupport@quidel.com](mailto:technicalsupport@quidel.com) or your local distributor.*

## CONFIGURING VIRENA


1. Before configuring Virena, log in to the **Supervisor Menu**, select **Settings**, then **Other Settings**.

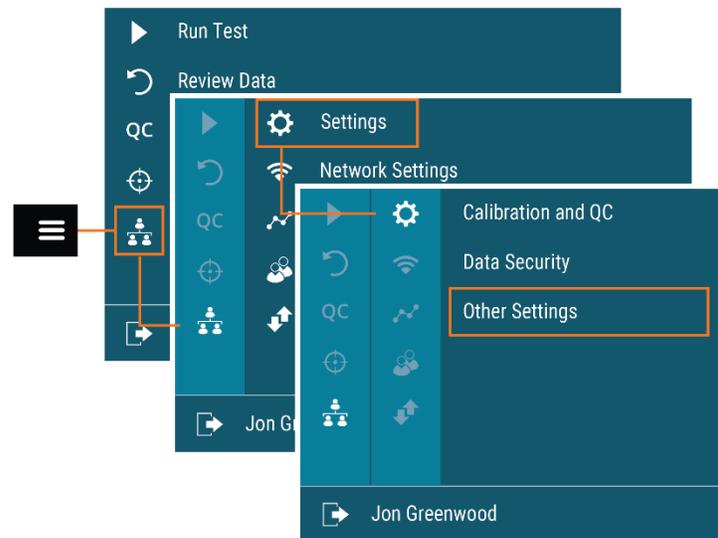


2. On the first screen, enter the 5-digit ZIP Code that was entered for the facility in MyVirena, select ✓ to confirm the entry and close the keyboard, then select > to advance to the next screen. Continue to set remaining settings as needed and return to the Run Test screen.

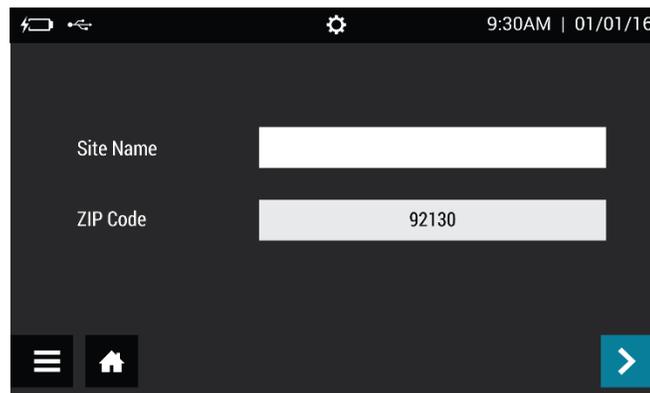


3. Returning to the **Supervisor Menu**, select **Network Settings**, then **Virena Settings**.

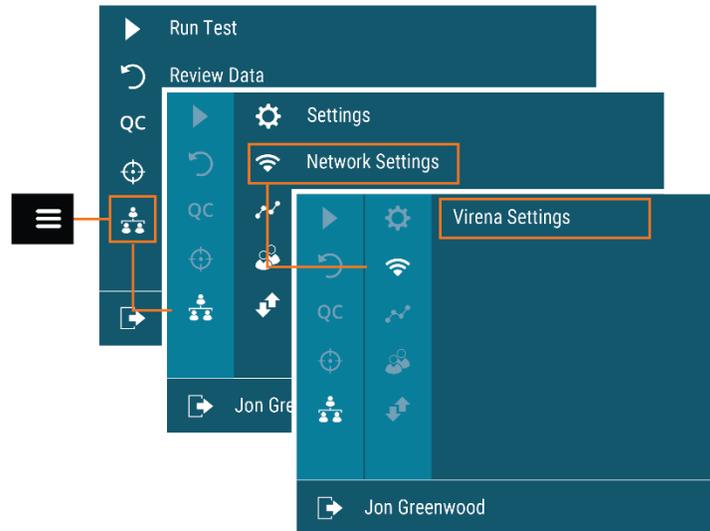


4. On the first screen, confirm that **Virena Domain Name** is pre-populated as **my.devicecloud.com**. If not, re-enter the information as shown below. Select **>** to advance to the next screen.

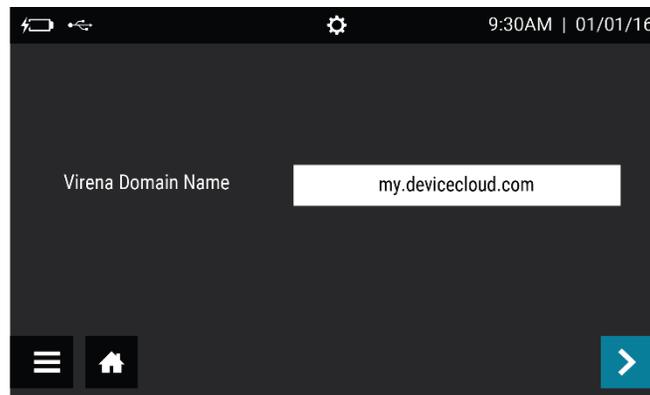


5. Set **Auto Send** and **Queue Results** to **On**. Select **>** to advance to the next screen.

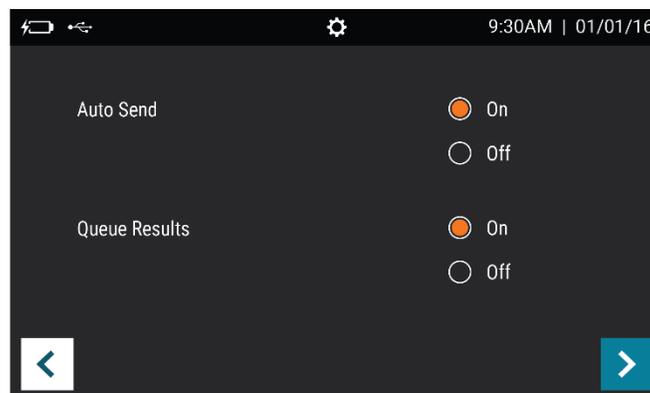

6. Confirm that **Transmit Patient ID** and **Transmit Order Number** are set to **Off**. Select **>** to advance to the next screen.

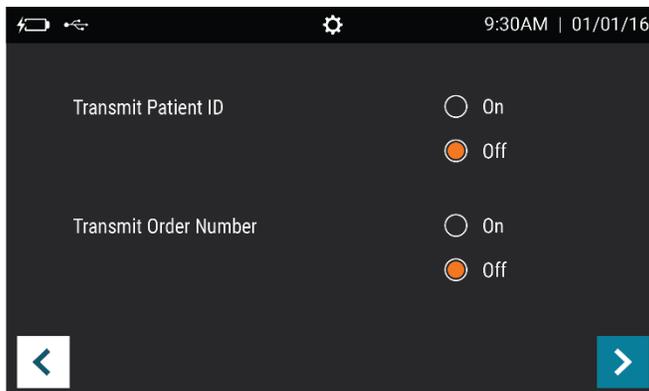


7. Set **Age Required** to **Yes**. Select **✓** to return to the Run Test Screen.

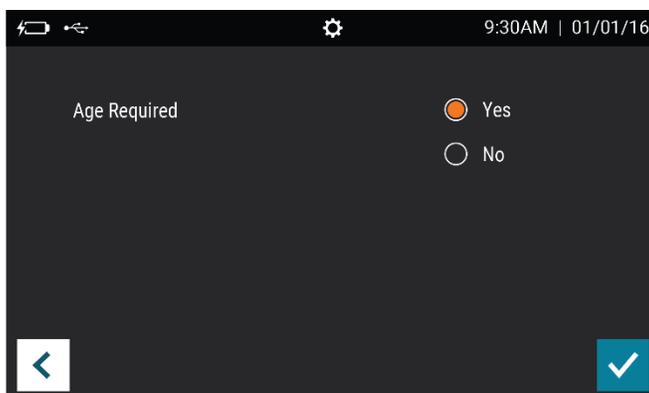


## TESTING THE VIRENA CONNECTION


8. From the **Supervisor Menu**, select **Export, Import, Delete**.

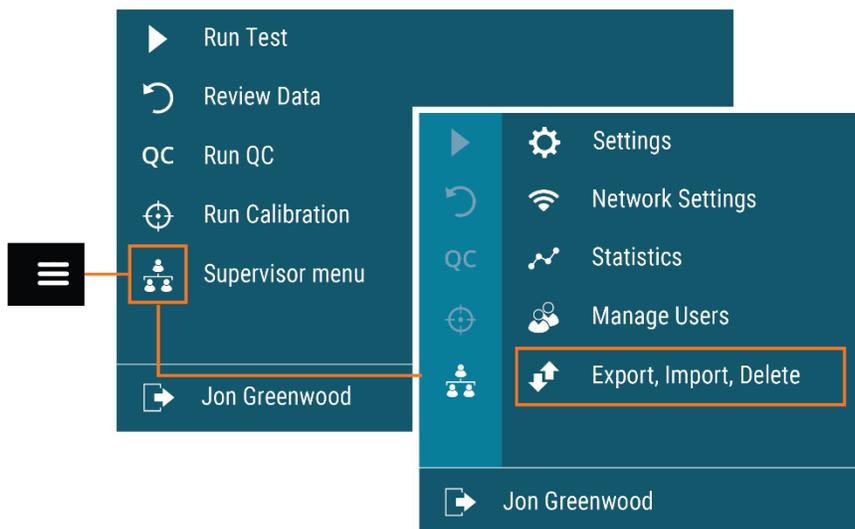

9. In the top scroll bar, select  until **Test Connection** appears. Select  to begin testing the connection.

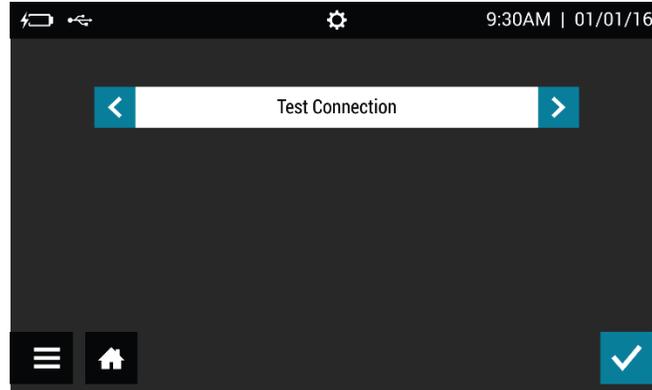

10. If the connection is successful, the message **Test connection success. Signal strength -##dBm.** will appear. Sofia 2 is now connected to the Virena cloud. If the connection is unsuccessful, contact Technical Support.

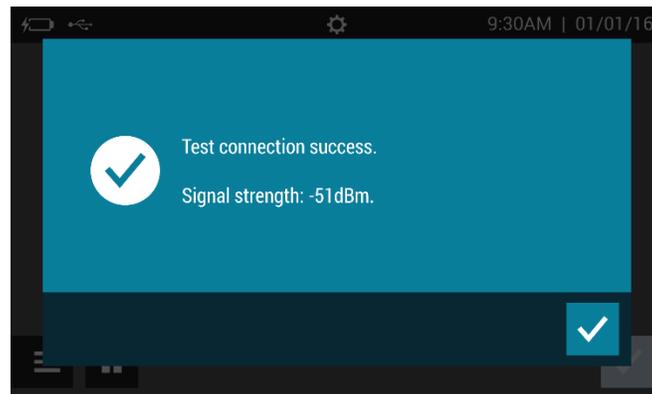

11. Run Calibration, Patient Test, or QC to send a result to the Virena cloud and confirm the connection.

