Contents

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Software Download from Document Library

Introduction

Siemens Healthcare Diagnostics is introducing the ability to download system software from the Siemens Healthineers Document Library onto a blank USB stick. You can then use the USB stick to install software upgrades onto your system.

Note: You'll need a blank 2, 4, 8, or 16 GB USB stick formatted to FAT32.

Performance testing of this process was performed on SanDisk branded USBs.

You must be a registered user to login and access the Document Library. If you don't have a login and password, use the link below and follow the instructions before downloading system software.

<https://www.siemens-healthineers.com/services/laboratory-diagnostics/service-and-support/technical-documentation>

USB Stick Format Instructions

To format your USB stick, perform the following steps:

1. Insert the USB stick into your computer.
2. Open File Explorer.
3. Right click on your USB stick.
4. Select **Format**.
5. Open the **File System** dropdown menu and select **FAT32**.

Note: Make sure Allocation unit size is set to default.

6. Click **Start**.
7. Once done, click close to go back to your File Explorer folder.

Download Instructions

1. Open a web browser on your computer.
2. In the browser go to: <https://doclib.siemens-healthineers.com/home>
3. Log on to the Document Library using your email address and password.
4. In the Document Library Search bar, do one of the following:
 - To search for a specific system, type the name of the system (for example, DCA Vantage 4.5) and click **Enter**.
 - To search for all system updates, type **Software Updates** and click **Enter**.

A new page opens, displaying a list of documentation and software upgrade files matching your search entry.

Note: Search results display in English or your profile default language setting.

5. In the search results list, click the checkbox for the software update you want to download.
6. Insert the blank formatted USB stick into a USB slot on your computer.
7. Click the **Download** button to begin downloading the software upgrade to the download folder of your system. Be sure to record the location of the ZIP file.
8. Unzip the download and copy the files directly onto the formatted USB stick.
9. Once complete, refer to the *Installation Instructions* for your system to install the upgrade.

Note: Contact your Siemens Healthineers support representative if you have any issues with the download or upgrade installation.

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