DCA Vantage®
Analyzer

Self-test Use

Siemens Healthcare Diagnostics Inc.

10845186 Rev. A

2013-08

Unpacking Guide

Introduction

Use this guide to unpack your DCA Vantage® analyzer. For complete installation and configuration instructions, refer to your *DCA Vantage Operator's Guide, Section 1, Installation*.

Procedure

Your DCA Vantage analyzer is delivered in 1 shipping carton.

1. Carefully remove the contents of the shipping carton.
2. Inspect the carton and analyzer for visible signs of damage.
3. If damage to the analyzer exists, immediately file a complaint with the carrier.
4. Make sure all items listed below are included with your analyzer, and keep them for future use.
 - DCA Vantage analyzer
 - Cleaning Kit
 - Operator's Guide
 - Optical Test Cartridge (in system cartridge compartment)
 - European Power Cord for System
 - Replacement Fuse
 - Air Filter Replacement Kit
 - HbA_{1c} Quick Reference Guide
 - Documentation CD (Operator's Guide)
 - Paper Roll
 - Self Adhesive Label Roll
5. Retain the DCA Vantage shipping carton and packing.

If you need to ship the analyzer, the shipping carton affords the best protection against damage.

6. Install and configure the analyzer:
 - a. Ensure that the system power switch is in the off position.
 - b. Connect the power cord to the system and to an appropriate grounded AC electrical outlet.

Starting the DCA Vantage System

To start the DCA Vantage system, turn the power button to the On position. The system displays the following information:

- Model Name
- System Software Version
- Copyright Information

Loading Configuration Settings

The DCA Vantage provides a setup wizard to assist in loading configuration settings. You can use the setup wizard to load configuration settings. For more information refer to your *DCA Vantage Operator's Guide, Section 1 Installation, Using the System Settings Wizard*.

Trademark Information

DCA Vantage is a trademark of Siemens Healthcare Diagnostics.

DCA Vantage®

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Per uso autodiagnostico

Siemens Healthcare Diagnostics Inc.

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